

Please contact Customer Service  
with any questions:

(877) 236-4408

**PRECISE<sup>®</sup>** **LTM**  
Diode Laser

## Fiber Replacement Instructions

---



 **CAO GROUP, INC.**  
Easier · Faster · Better™

[www.caogroup.com/dental](http://www.caogroup.com/dental)

P/N 03844 Rev 1 21DEC2015

# 1

Place unit upside down on a hard surface.



# 2




Slide fiber cartridge out and gently place on hard surface next to the unit.

Unscrew the connector to the empty cartridge and dispose.

# 3

Place the new cartridge on its side in place of the empty cartridge.

Grasp the metal ring at the base of the coupling to stabilize and prevent breakage.

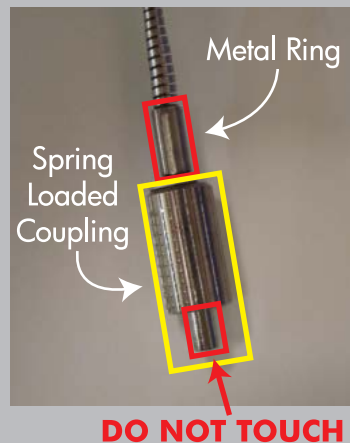

# 4



Push the coupling into the connection while maintaining your grasp on the metal ring.

Continue to hold metal ring and twist coupling into place with the other hand.

**Warning: If metal ring is left to twist, hyper-rotation of the fiber may occur resulting in irreparable damage.**

# 5



Gently guide fiber back into recessed space within the fiber cartridge.

Make sure the coil retracts into the cartridge as it is gently slid into place.